


Intended use

The EndoFender® is intended for the protection of the Bending Section Rubber of flexible endoscopes with a diameter between 5 and 17 mm, and keeps the valves as a unique set with the parent endoscope.


A

- 1 Valves to be placed in any of the two positions.
- 2 Distal cap can be placed here.
- 3 Close the EndoFender®; the button is on position I, place it inside the basin of the washer-disinfector next to the flexible endoscope.

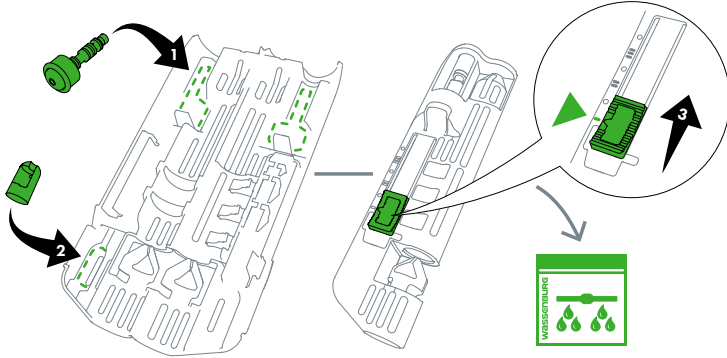


B

- 1 After the cleaning and disinfecting cycle inside the washer-disinfector, slide the button to the next position to open the EndoFender®. Place the endoscope inside.
- 2 Close the EndoFender® again. The button is on position II; The EndoFender® is attached to the clean and disinfected endoscope during drying/storage or transport.

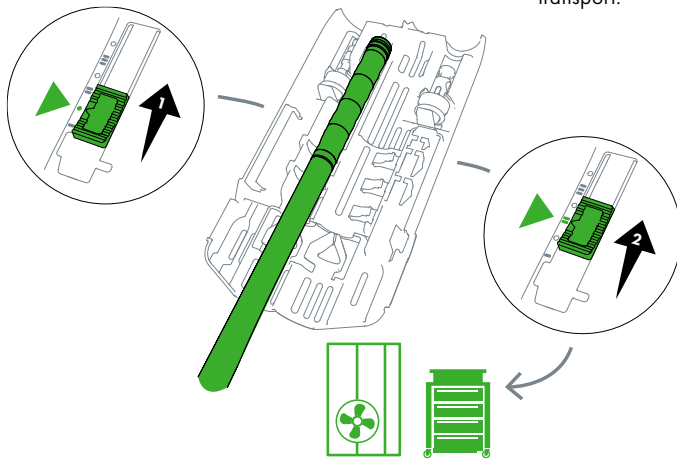


C

- 1 Slide the button to open the EndoFender® for the second time. Take out the endoscope.
- 2 Press down the pin to release the valve.

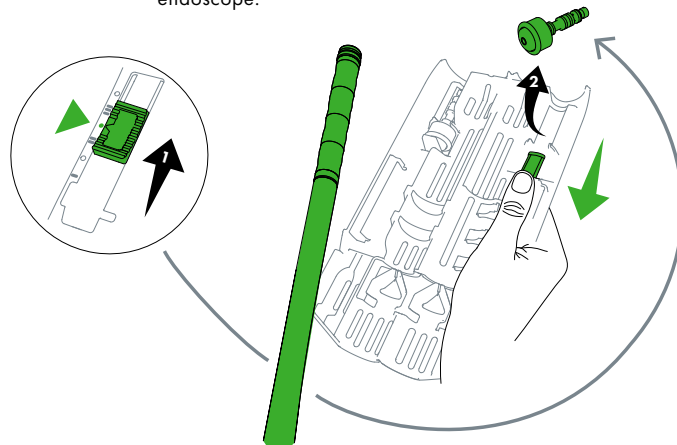

D

- 1 After procedure the valves stay inside the endoscope, only the contaminated endoscope is placed inside the EndoFender® to protect the tip and the Bending Section Rubber during transport.
- 2 Close the EndoFender®, the button is on position III.

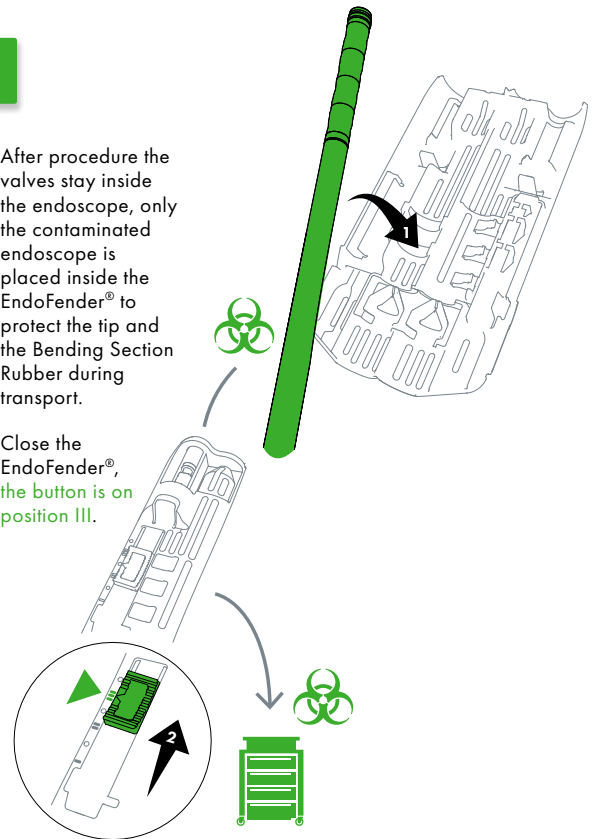

E

- 1 Slide the button once more to open the EndoFender® for the last time at the manual precleaning stage.
- 2 Take out the endoscope to be reprocessed. Dispose of EndoFender® in accordance with local regulations.

